

Assembly Instructions for the ASHTON GROUP FACE SHIELD



For each shield, your kit should include:

- 1 face shield
- 1 elastic strap
- 1 long polypropylene strap
- 1 short polypropylene strap


These are the polypropylene straps

The long strap features slots A (blue) and B (red), and tabs C (orange)

The short strap has tab D at either end (green)



Insert tab D into slots B

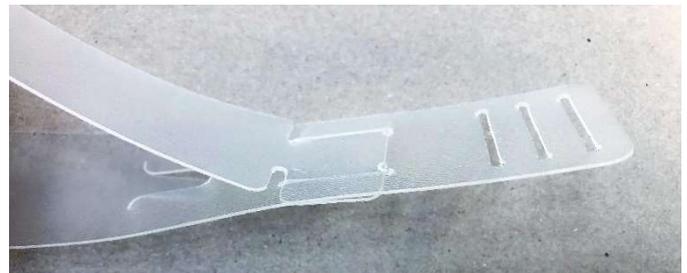


Repeat this on the other side



The straps should now look like this




Feed the ends of the elastic strap through slots A at one end of the long polypropylene strap as shown in the photo

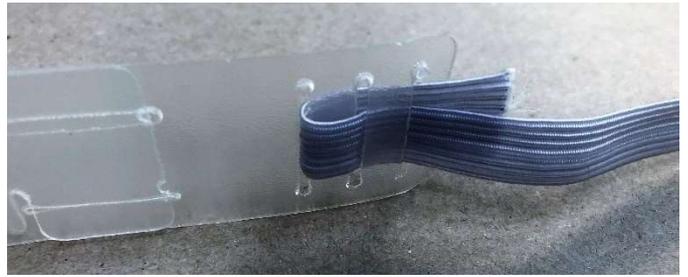

Repeat at the other side



Feed tabs C through the holes on the face shield

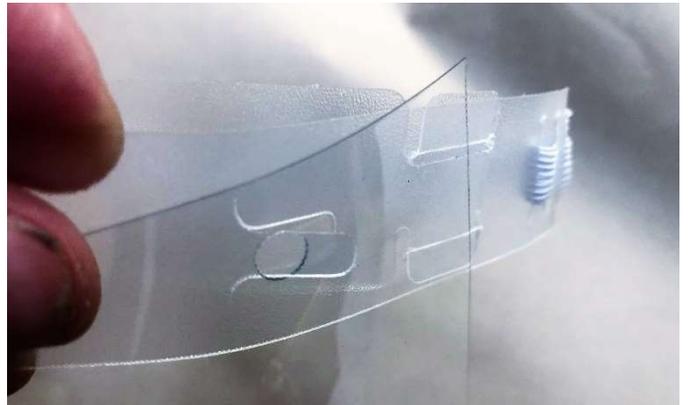

Your face shield should now look like this and is ready to use

